

LEAD APRON CARE & MAINTENANCE

Required Practices for Staff and Students

| DO ✓ | DON'T ✗ |
|--|---|
|  <p data-bbox="396 1205 863 1264">Wear correct size</p> |  <p data-bbox="1188 1205 1745 1264">Do not fold or crease</p> |
|  <p data-bbox="279 1700 978 1760">Hang on approved hanger</p> |  <p data-bbox="1058 1709 1871 1768">Do not sit or lean while wearing</p> |
|  <p data-bbox="243 2184 1020 2243">Clean with mild soap & water</p> |  <p data-bbox="1188 2139 1755 2273">Do not machine wash or autoclave</p> |
|  <p data-bbox="373 2659 884 2718">Inspection regularly</p> |  <p data-bbox="1094 2659 1843 2718">Do not use damaged aprons</p> |

PROPER CARE, PROPER PROTECTION

**Radiology Department
Hospital Pakar Sultanah Fatimah**

(Untuk kegunaan dalaman sahaja / For
internal use only)

PENJAGAAN BAJU PERLINDUNGAN SINARAN PLUMBUM

CARE OF LEAD RADIATION PROTECTIVE GARMENTS

Apron • Thyroid Shield • Gonad Shield



PENYIMPANAN STORAGE



- Gantung secara menegak pada hanger khas
(Hang vertically using a dedicated hanger)
- Jangan dilipat atau disangkut separuh
(Do not fold or hang partially)
- Simpan di tempat kering dan tidak lembap
(Store in a dry, non-humid area)

PENGGUNAAN USAGE

- Pakai dengan kemas dan cermat
(Wear properly and carefully)
- Elakkan duduk, bersandar atau memijak apron
(Do not sit, lean on, or step on the apron)
- Gunakan saiz yang sesuai
(Use the appropriate size)



PEMERIKSAAN BERKALA REGULARLY INSPECTION

- Pemeriksaan X-ray / Fluoroscopy sekurang-kurangnya setahun sekali
(X-ray / fluoroscopy inspection at least once a year)
- Periksa rekahan, koyakan atau penipisan
(Check for cracks, tears, or thinning)
- Laporkan segera jika rosak
(Report immediately if damaged)



AMARAN! WARNING!

- ✗ Lipatan boleh menyebabkan rekahan plumbum
(Folding may cause lead cracks)
- ✗ Apron rosak tidak memberi perlindungan optimum
(Damaged aprons do not provide adequate protection)

PEMBERSIHAN CLEANING

- Lap menggunakan kain lembap
(Wipe with a damp cloth)
- Gunakan sabun lembut sahaja
(Use mild soap only)
- Jangan rendam atau basuh mesin
(Do not soak or machinewash)



PROPER CARE, PROPER PROTECTION

**Radiology Department
Hospital Pakar Sultanah Fatimah**

(Untuk kegunaan dalaman sahaja / For internal use only)